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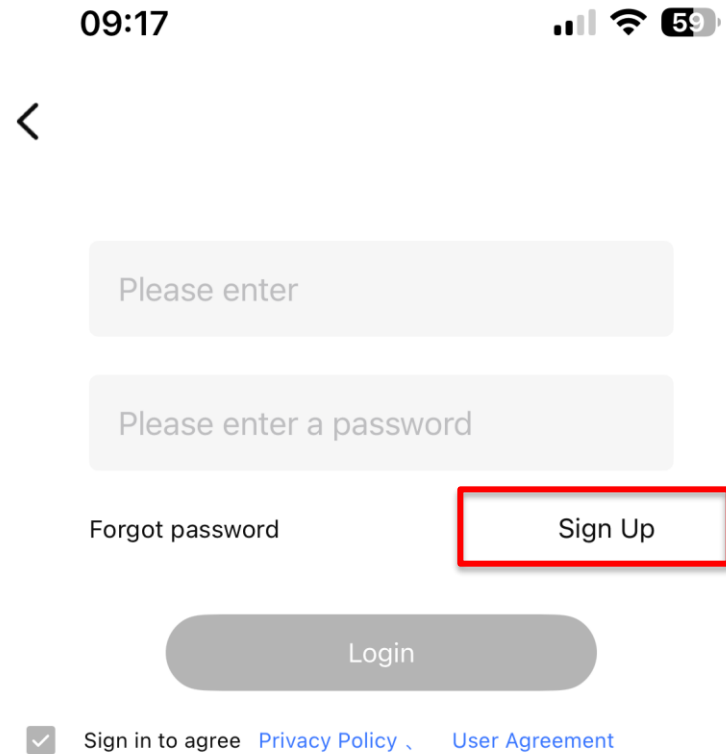
09:17

Signal strength, Wi-Fi, and battery (59%) icons.

<

Please enter

Please enter a password

Forgot password

Sign Up

Login

☒ Sign in to agree [Privacy Policy](#) [User Agreement](#)

The image shows a mobile application interface for user authentication. At the top, the time is 09:17, and there are status icons for signal strength, Wi-Fi, and a 59% battery level. Below the status bar is a back arrow icon. The main form contains two text input fields: the first is labeled 'Please enter' and the second is labeled 'Please enter a password'. Below these fields are two links: 'Forgot password' and 'Sign Up'. The 'Sign Up' link is highlighted with a red rectangular border. Below the links is a large, rounded 'Login' button. At the bottom of the screen, there is a checkbox that is checked, followed by the text 'Sign in to agree' and two links: 'Privacy Policy' and 'User Agreement'.

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Please enter

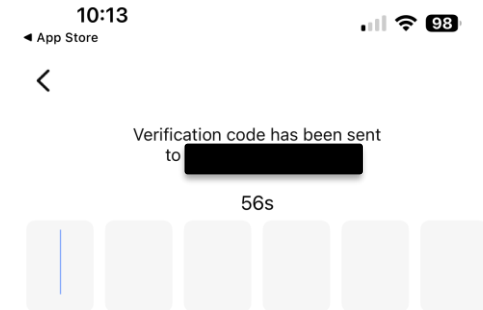
Get Verification Code

☒ Read in to agree [Privacy Policy](#) [User Agreement](#)

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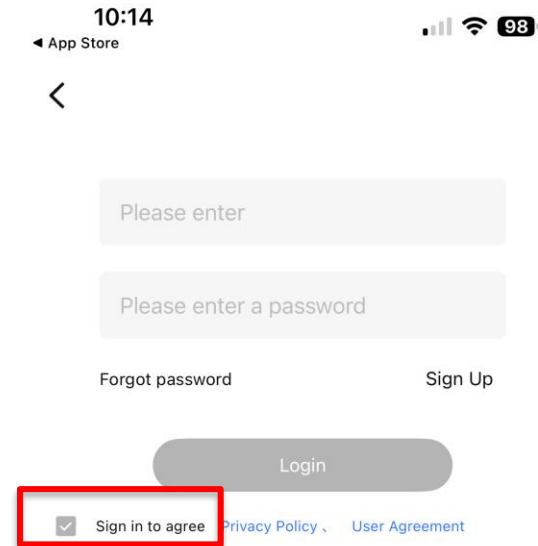
A screenshot of a mobile application's login screen. At the top, the status bar shows the time 10:14, signal strength, Wi-Fi, and battery at 98%. Below the status bar is a navigation bar with a back arrow and the text 'App Store'. The main content area contains two text input fields: the first is labeled 'Please enter' and the second 'Please enter a password'. Below these fields are two links: 'Forgot password' and 'Sign Up'. A large, rounded 'Login' button is centered below the links. At the bottom, there is a checkbox labeled 'Sign in to agree' which is checked with a blue checkmark. To the right of this checkbox are two links: 'Privacy Policy' and 'User Agreement'.

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